


# Achieve unparallelled ureteral access.

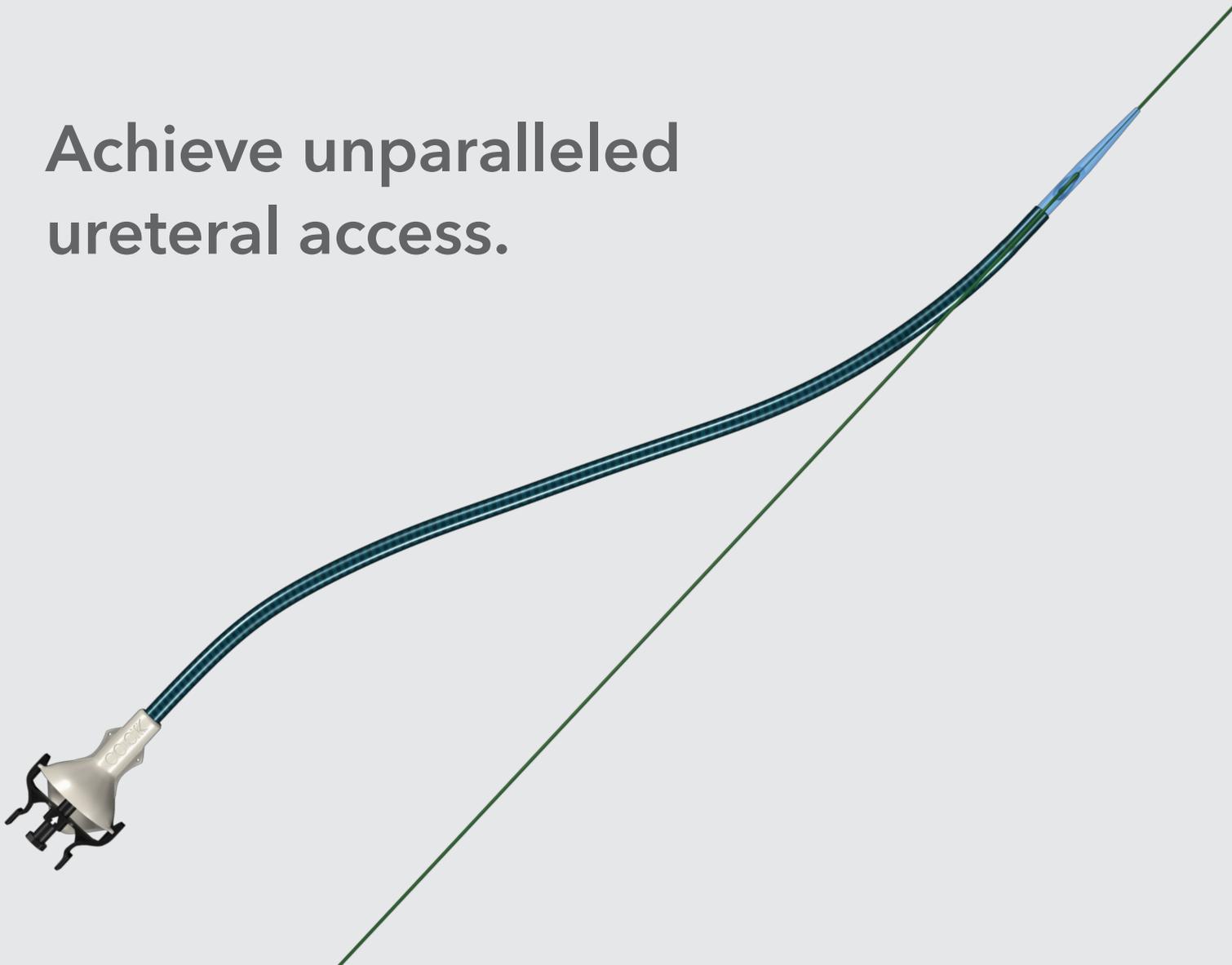


## Flexor® Parallel™

RAPID RELEASE™ URETERAL ACCESS SHEATH

Cook Medical's Flexor® Parallel™ Rapid Release™ Ureteral Access Sheath effectively alters the way in which you achieve and maintain access. With Flexor Parallel you have the option to insert the sheath using either the rapid-release technique, which eliminates the need for a second wire, or the traditional Flexor placement method.



[cookmedical.com](http://cookmedical.com)

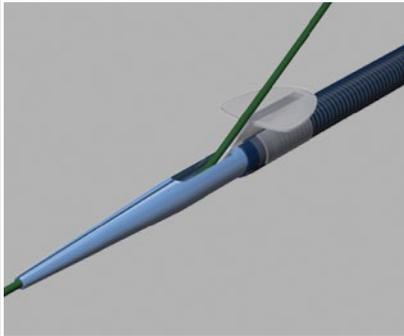


## Flexor® Parallel™

RAPID RELEASE™ URETERAL ACCESS SHEATH

When using the rapid-release technique, advance the Flexor Parallel over a previously placed wire guide through the opening in the dilator. Once the sheath is in place, remove the dilator. This causes the wire guide to rapidly release from the dilator tip and transition outside the sheath. This technique provides a free working channel and a safety wire to maintain the tract.

The rapid-release technique eliminates a step from the procedure, saves product, and streamlines inventory.

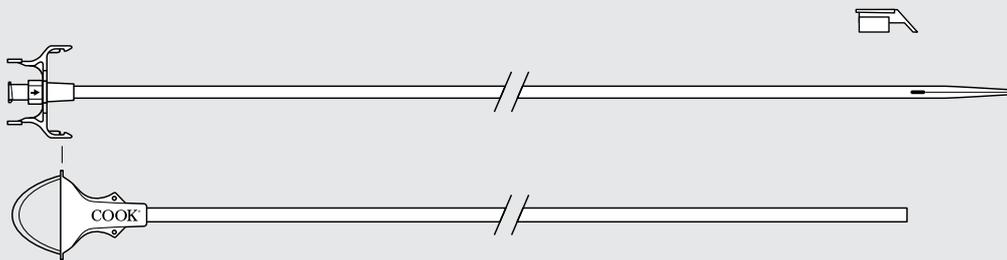

Order Number	Reference Part Number	Internal Diameter Fr	Length cm	Order Number	Reference Part Number	Internal Diameter Fr	Length cm
G56428	FUS-095020-P	9.5	20	G56433	FUS-120020-P	12.0	20
G56429	FUS-095028-P	9.5	28	G56434	FUS-120028-P	12.0	28
G56430	FUS-095035-P	9.5	35	G56435	FUS-120035-P	12.0	35
G56431	FUS-095045-P	9.5	45	G56436	FUS-120045-P	12.0	45
G56432	FUS-095055-P	9.5	55	G56438	FUS-120055-P	12.0	55
G34871	FUS-107020-P	10.7	20	G56440	FUS-140028-P	14.0	28
G34872	FUS-107028-P	10.7	28	G56441	FUS-140035-P	14.0	35
G34873	FUS-107035-P	10.7	35	G56442	FUS-140045-P	14.0	45
G34874	FUS-107045-P	10.7	45	G56443	FUS-140055-P	14.0	55
G34875	FUS-107055-P	10.7	55				

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support for details.

### Customer Service

EU Website: [cookmedical.eu](http://cookmedical.eu)  
 EDI: [cookmedical.eu/edi](http://cookmedical.eu/edi)  
 Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
 Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
 Belgium: +32 27001702, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
 Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
 Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
 France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
 Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
 Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
 Poland: +48 223060159, [pl.orders@cookmedical.com](mailto:pl.orders@cookmedical.com)  
 Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
 Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
 Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

USA Website: [cookmedical.com](http://cookmedical.com)  
 EDI: [cookmedical.com/edi.do](http://cookmedical.com/edi.do)  
**Americas:**  
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
 E-mail: [customersupport@cookmedical.com](mailto:customersupport@cookmedical.com)  
**Australia:**  
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283  
 E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



CC, URO-A4