Band ligation-assisted endoscopic mucosal resection in the UGI tract

Duette®

MULTI-BAND MUCOSECTOMY

By combining a multi-band ligator with a snare, Duette facilitates targeted tissue acquisition and resection in the upper gastrointestinal (UGI) tract. Duette's inventive design can help you achieve positive outcomes in your endoscopic mucosal resection procedures.





Procedure photo credit: Robert Enns, MD, FRCP, and Sunny Lee, MBBS, FRACP

Duette

MULTI-BAND MUCOSECTOMY

Used for endoscopic mucosal resection in the UGI tract. The snare component is supplied sterile, and the banding component is supplied nonsterile. This product is intended for single use only.

- The device setup is similar to that of the 6 Shooter® Saeed® Multi-Band Ligator.
- The six latex rubber bands are available for resection purposes. A single clear band is included on the barrel as an alert that only one additional band remains for use.
- The snare wire is soft, braided and a mini-hexagonal configuration.
- The snare is available in 5 and 7 Fr.
- The snare may be passed through the ligator handle, allowing for speedy resection of ligated tissue without the removal of the endoscope or the disassembly of the components.

Order Number	Reference Part Number	Endoscope Outer Diameter mm	Number of Bands	Sheath Fr	Snare Size cm	Minimum Accessory Channel Diameter mm
G34036	DT-6	9.5–13	6	7	1.5 x 2.5	3.7
G35129	DT-6-5F	9.5–13	6	5	1.5 x 2.5	2.8
G34054	DT-6-XL	11–14	6	7	1.5 x 2.5	3.7

 $\textbf{\textit{CAUTION}}: This \ product \ contains \ natural \ rubber \ latex, \ which \ may \ cause \ an \ allergic \ reaction.$

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support & Delivery for details.

Customer Service

EU Website: cookmedical.eu EDI: cookmedical.eu/edi

Distributors: +353 61239240, ssc.distributors@cookmedical.com

Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001702, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Iceland: +354 800 7615, is.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com

Switzerland - French: +41 448009609, fr.orders@cookmedical.com Switzerland - Italian: +41 448009609, it.orders@cookmedical.com Switzerland - German: +41 448009609, de.orders@cookmedical.com United Kingdom: +44 2073654183, uk.orders@cookmedical.com USA Website: cookmedical.com EDI: cookmedical.com/edi.do

Americas:

Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335

E-mail: customersupport@cookmedical.com

Australia:

Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283

E-mail: cau.custserv@cookmedical.com



