


# Band ligation-assisted endoscopic mucosal resection in the UGI tract

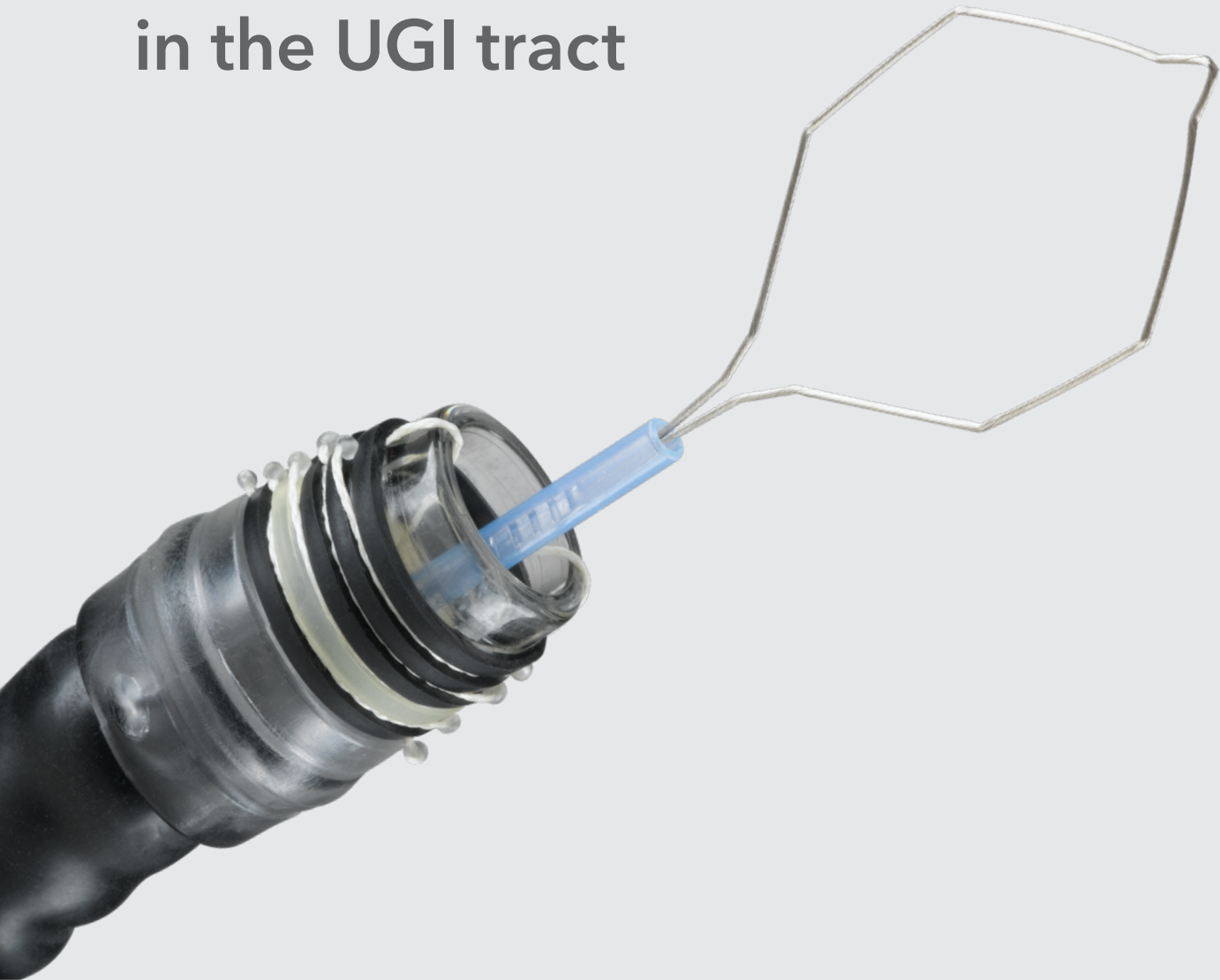


Duette®

MULTI-BAND MUCOSECTOMY

By combining a multi-band ligator with a snare, Duette facilitates targeted tissue acquisition and resection in the upper gastrointestinal (UGI) tract. Duette's inventive design can help you achieve positive outcomes in your endoscopic mucosal resection procedures.



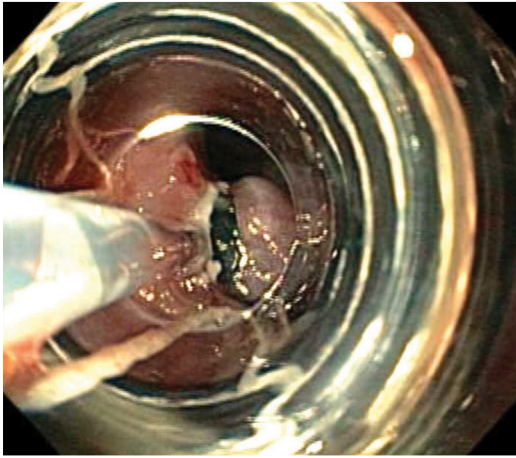

Procedure photo credit: Robert Enns, MD, FRCP, and Sunny Lee, MBBS, FRACP

## Duette®

MULTI-BAND MUCOSECTOMY

Used for endoscopic mucosal resection in the UGI tract. The snare component is supplied sterile, and the banding component is supplied nonsterile. This product is intended for single use only.

- The device setup is similar to that of the 6 Shooter® Saeed® Multi-Band Ligator.
- The six latex rubber bands are available for resection purposes. A single clear band is included on the barrel as an alert that only one additional band remains for use.
- The snare wire is soft, braided and a mini-hexagonal configuration.
- The snare is available in 5 and 7 Fr.
- The snare may be passed through the ligator handle, allowing for speedy resection of ligated tissue without the removal of the endoscope or the disassembly of the components.

Order Number	Reference Part Number	Endoscope Outer Diameter mm	Number of Bands	Sheath Fr	Snare Size cm	Minimum Accessory Channel Diameter mm
G34036	DT-6	9.5–13	6	7	1.5 x 2.5	3.7
G35129	DT-6-5F	9.5–13	6	5	1.5 x 2.5	2.8
G34054	DT-6-XL	11–14	6	7	1.5 x 2.5	3.7

**CAUTION:** This product contains natural rubber latex, which may cause an allergic reaction.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support & Delivery for details.

Refer to the Instructions for Use (IFU) for complete product information and full prescribing information including indications, contraindications, warnings, precautions, and potential adverse events. Please see product risk information in the IFU at [www.cookmedical.eu](http://www.cookmedical.eu)

### Customer Service

EU Website: [cookmedical.eu](http://cookmedical.eu)  
 EDI: [cookmedical.eu/edi](http://cookmedical.eu/edi)  
 Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
 Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
 Belgium: +32 27001702, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
 Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
 Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
 France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
 Iceland: +354 800 7615, [is.orders@cookmedical.com](mailto:is.orders@cookmedical.com)  
 Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
 Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
 Norway: +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)  
 Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
 Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
 Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

USA Website: [cookmedical.com](http://cookmedical.com)  
 EDI: [cookmedical.com/edi.do](http://cookmedical.com/edi.do)  
**Americas:**  
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
 E-mail: [customersupport@cookmedical.com](mailto:customersupport@cookmedical.com)  
**Australia:**  
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283  
 E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



AI-ESC-IR-OHNS-PI-RH-SUR-A4