
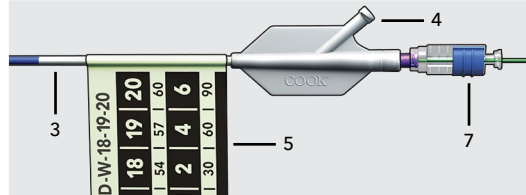
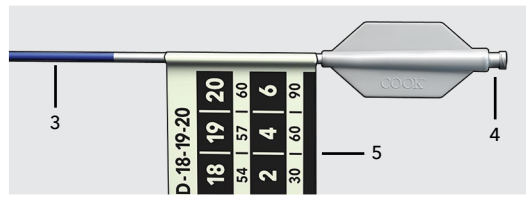
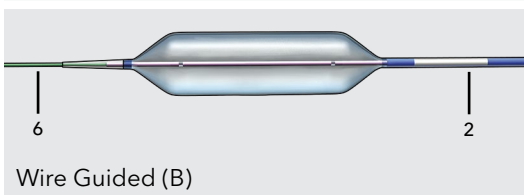


# Hercules®

3 STAGE BALLOON

## Features

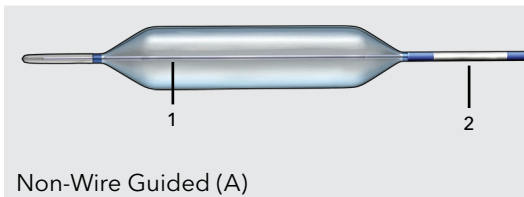

- 1) Rapid deflation sleeve
- 2) Silver positioning indicator
- 3) Express evacuation catheter
- 4) Inflation hub


- 5) Glow-in-the-dark catheter tag
- 6) 300 cm wire guide
- 7) Wire lock

## Step 1a




Fill the 60 mL inflation device with water. Purge excess air and water from the syringe by tilting the syringe assembly so that air bubbles float toward the Luer lock end, then pump the handle lever until all excess air and water is purged from syringe and connected tubing and the level of remaining water is 40 mL.

## Step 1b

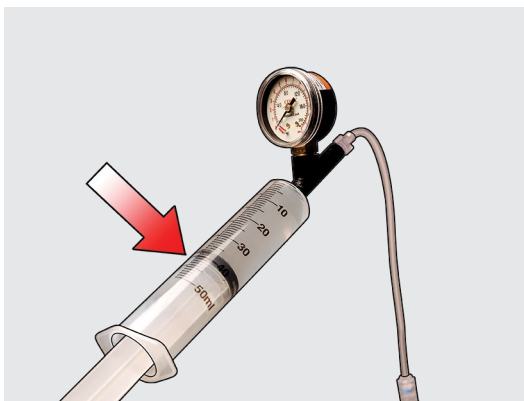


Maintain water level at the 40 mL line in the syringe as shown.

## Step 2

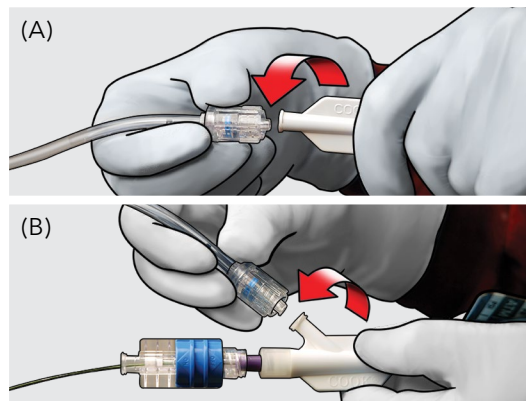


Securely attach the balloon inflation port to the inflation device.

## Step 3

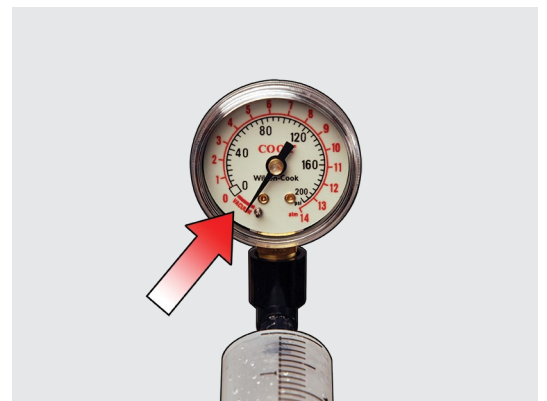


Apply negative pressure.

## Step 4

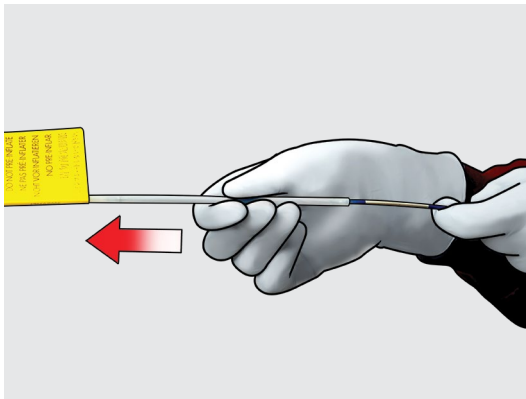


Remove the protective sheath from the balloon.

## Step 5

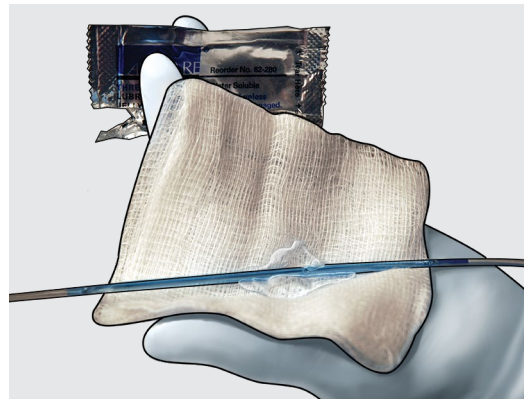


Lubricate the balloon completely. **Do not** pre-inflate the balloon prior to introduction into the endoscope. After visualization of the silver positioning indicator, advance the balloon to the desired position.

## Step 5.5 (Optional)

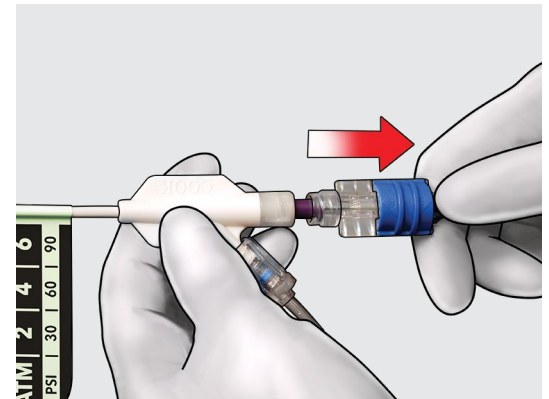


**(Wire guided balloon only)**  
To advance the wire guide in the wire guided version, move the blue wire lock to the "ON" position.

## Step 6

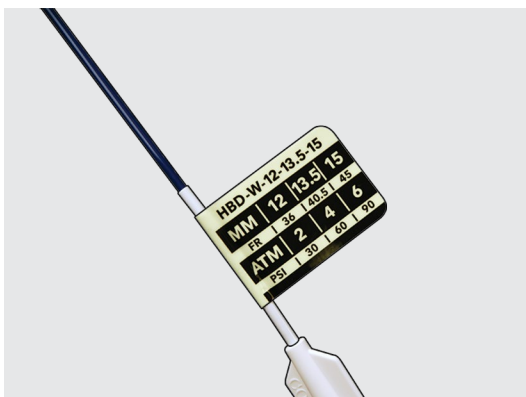

Increase pressure to the specified pressure levels shown on the catheter tag.

## Step 7

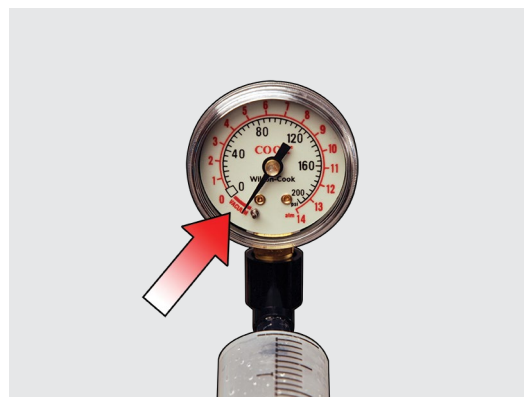

For deflation, apply negative pressure. Remove the balloon from the scope after the balloon is deflated.

Reference Part Number	Order Number
HBD-8-9-10	G31928
HBD-10-11-12	G31925
HBD-12-13.5-15	G31926
HBD-15-16.5-18	G48732
HBD-18-19-20	G31927
HBD-W-8-9-10	G51799
HBD-W-10-11-12	G51795
HBD-W-12-13.5-15	G51796
HBD-W-15-16.5-18	G51797
HBD-W-18-19-20	G51798

**Refer to the current Instructions for Use for detailed system use.**

**Please see product risk information in the IFU at [cookmedical.eu](http://cookmedical.eu).**

### Customer Service

EU Website: [cookmedical.eu](http://cookmedical.eu)  
 EDI: [cookmedical.eu/edi](http://cookmedical.eu/edi)  
 Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
 Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
 Belgium: +32 27001702, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
 Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
 Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
 France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
 Iceland: +354 8007615, [IS.orders@cookmedical.com](mailto:IS.orders@cookmedical.com)  
 Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
 Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
 Norway: +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)  
 Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
 Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
 Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

USA Website: [cookmedical.com](http://cookmedical.com)  
 EDI: [cookmedical.com/edi.do](http://cookmedical.com/edi.do)  
**Americas:**  
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
 E-mail: [customersupport@cookmedical.com](mailto:customersupport@cookmedical.com)  
**Australia:**  
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283  
 E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



AI, ESC, IR, OHNS, PI, RH, SUR-8.5X11