

Get accurate results in **10 minutes** or less.

Transforming upper
GI bleed detection:
optimizing patient
management



PILLSENSE™
GI BLEED DETECTION SYSTEM



Rapid results that clearly make sense

The PillSense GI Bleed Detection System is the first FDA-approved ingestible blood-detection system that can rapidly and accurately detect upper GI bleeding using light absorption technology.

Use the PillSense Capsule to efficiently triage and prioritize patients with suspected GI bleeding and confirm bleeds in hemodynamically stable patients. It's designed to enhance your risk stratification and decision making for a more efficient use of critical resources.



Clinically proven accuracy:



1. Akiki K, Mahmoud T, Alqaisieh MH, et al. A novel blood-sensing capsule for rapid detection of upper GI bleeding: a prospective clinical trial. *Gastrointest Endosc.* 2024;99(5):712-720.

An assessment that's easy to swallow

The PillSense Capsule is readily available and easy to use. There's no need to prepare the patient, and it can be administered by a nurse or tech without any specialized training. The patient simply swallows the PillSense Capsule, and real-time diagnostic results are transmitted to the handheld PillSense Receiver in under 10 minutes—displayed with the message: "Blood Detected" or "No Blood Detected." No image interpretation is needed.

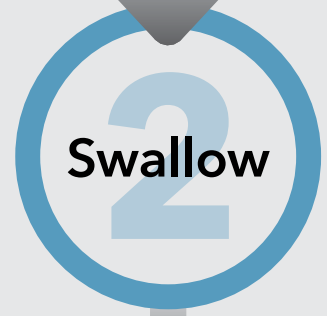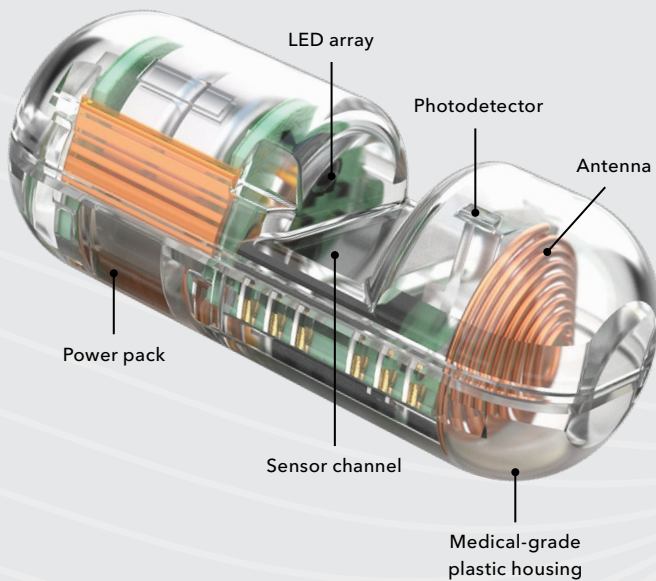

The **PillSense GI Bleed Detection System** is a prescription-only device consisting of a reusable receiver and a single-use ingestible capsule, intended to be used for the detection of blood in the upper gastrointestinal tract in hemodynamically stable adults suspected of having upper gastrointestinal bleeding (UGIB).

The PillSense Capsule is for single-use only and is provided nonsterile. The PillSense Receiver is a reusable component and is also provided nonsterile.

PillSense Capsule		PillSense Receiver	
The noninvasive ingestible capsule features an optical sensor to detect blood, and results are wirelessly transmitted to the receiver.		Results from the capsule are displayed in a simple and definitive message: "Blood Detected" or "No Blood Detected."	
Dimensions	11 x 27 mm	Dimensions	176.5 x 142.5 mm
Material	Biocompatible plastic	Material	Medical-grade plastic
Battery life	Minimum 6 hours	Battery life (rechargeable)	Minimum 6 hours at full charge
Blood-sensing technology	Multiple LEDs at different wavelengths and one optical sensor		

	Order Number	Reference Part Number	Box Quantity
PillSense Capsule	G60326	EPS-01-C	10
PillSense Receiver	G60327	EPS-01-R	2

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support & Distribution for details.

Please refer to the product's Instructions for Use (IFU) for full prescribing information, warnings, precautions, contraindications, and potential adverse events.

Manufactured by EnteraSense Ltd.
PillSense is a trademark of EnteraSense Ltd.

Customer Service

EU Website: cookmedical.eu
 EDI: cookmedical.eu/edi
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001702, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Iceland: +354 800 7615, is.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

USA Website: cookmedical.com
 EDI: cookmedical.com/edi
Americas:
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: customersupport@cookmedical.com
Australia:
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
 E-mail: cau.custserv@cookmedical.com



AI-ESC-IR-OHNS-PI-RH-SUR-A4