


# Accurate, consistent resection

Control every angle  
with rotatability  
to ensure efficient  
polyp removal.

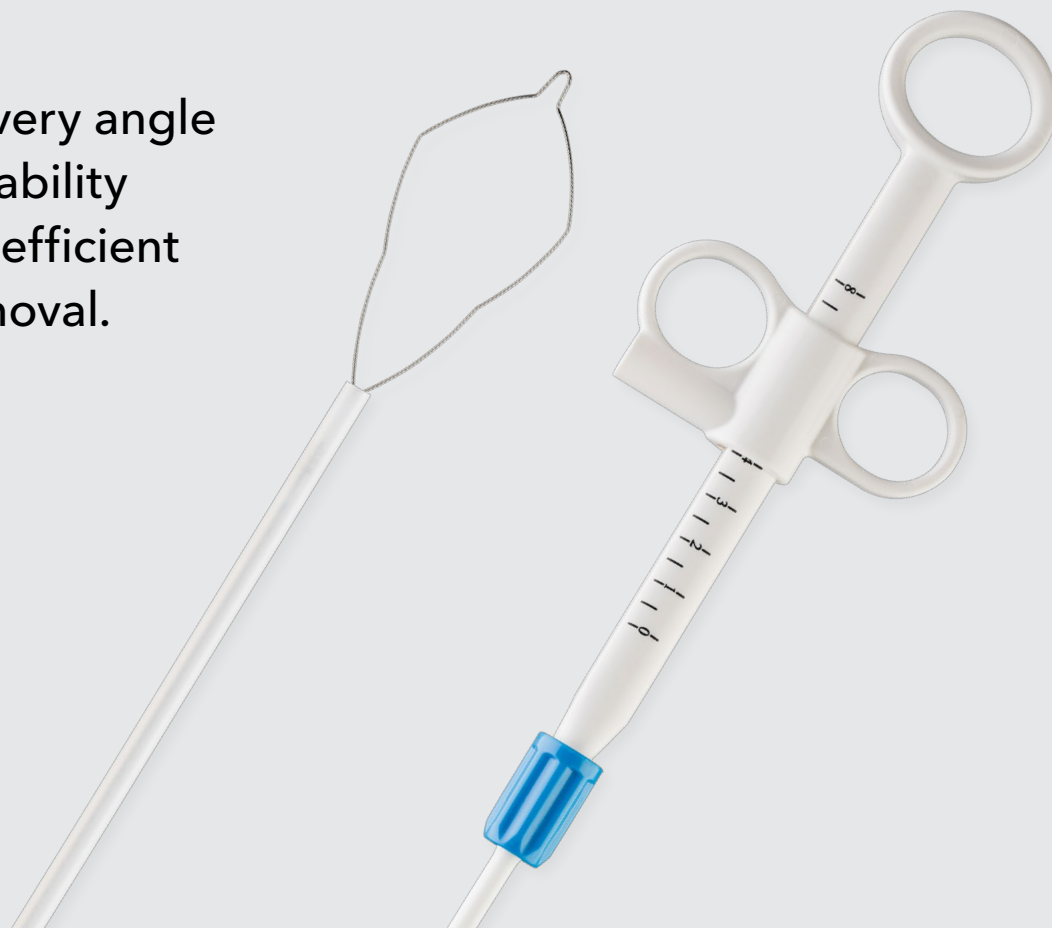

TISSUE MANAGEMENT

TARGET | ACQUIRE | TREAT

## Cold AcuSnare®

POLYPECTOMY SNARE

### Precise snare placement


The real-time, bidirectional, rotatable design enables you to resect polyps accurately and efficiently. The ergonomic and flexible handle is designed to maximize ease of use and streamline workflow.

### Designed for cold snare polypectomy

Due to its ability to withstand a high level of pulling and stretching force, the loop wire enables you to confidently perform a controlled and precise cold snare polypectomy for diminutive and small polyps.

### Bend-resistant wire loop

The loop is composed of bend-resistant, multifilament stainless steel wire, which allows you to make multiple cuts without the loop losing its shape.

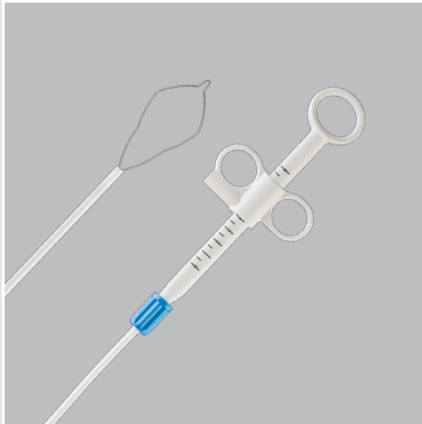

## Cold AcuSnare®

### POLYPECTOMY SNARE

The Cold AcuSnare Polypectomy Snare is intended to be used without diathermic energy for the endoscopic resection of diminutive and small polyps in the gastrointestinal tract.

It has multiple product specifications and models suitable for a variety of clinical needs.

It is supplied sterile and disposable. It is intended for single use only.

Order Number	Reference Part Number	Sheath Diameter mm	Working Length cm	Snare Opening Size mm	Cutting Wire Diameter mm	Quantity
G61206	CACS-10-025	2.4	230	10	0.25	10
G61204	CACS-10-030	2.4	230	10	0.30	10
G61207	CACS-15-025	2.4	230	15	0.25	10
G61205	CACS-15-030	2.4	230	15	0.30	10

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support & Distribution for details.

### Customer Service

EU Website: [cookmedical.eu](http://cookmedical.eu)

EDI: [cookmedical.eu/edi](http://cookmedical.eu/edi)

Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)

Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)

Belgium: +32 27001702, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)

Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)

Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)

France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)

Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)

Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)

Iceland: +354 8007615, [IS.orders@cookmedical.com](mailto:IS.orders@cookmedical.com)

Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)

Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)

Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)

Norway: +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)

Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)

Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)

Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)

Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)

Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)

United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

USA Website: [cookmedical.com](http://cookmedical.com)

EDI: [cookmedical.com/edi.do](http://cookmedical.com/edi.do)

#### Americas:

Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335

E-mail: [customersupport@cookmedical.com](mailto:customersupport@cookmedical.com)

#### Australia:

Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283

E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



AI, ESC, IR, OHNS, PI, RH, SUR-8.5X11