

Achieve unparallelled ureteral access.




Flexor® Parallel™

RAPID RELEASE™ URETERAL ACCESS SHEATH

Cook Medical's Flexor® Parallel™ Rapid Release™ Ureteral Access Sheath effectively alters the way in which you achieve and maintain access. With Flexor Parallel you have the option to insert the sheath using either the rapid-release technique, which eliminates the need for a second wire, or the traditional Flexor placement method.



www.cookmedical.com

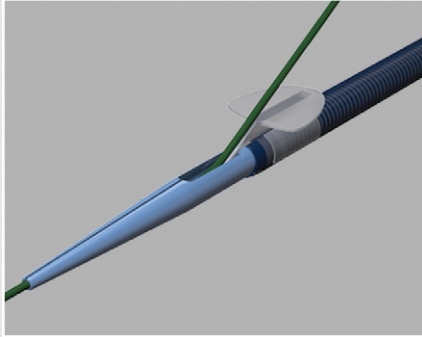

Flexor® Parallel™

RAPID RELEASE™ URETERAL ACCESS SHEATH

When using the rapid-release technique, advance the Flexor Parallel over a previously placed wire guide through the opening in the dilator. Once the sheath is in place, remove the dilator. This causes the wire guide to rapidly release from the dilator tip and transition to outside the sheath. This technique provides a free working channel and a safety wire to maintain the tract.

The rapid-release technique eliminates a step from the procedure, saves product and streamlines inventory.

Order Number	Reference Part Number	Internal Diameter Fr	Length cm	Order Number	Reference Part Number	Internal Diameter Fr	Length cm
G56445	FUS-095013-P	9.5	13	G56446	FUS-120013-P	12.0	13
G56428	FUS-095020-P	9.5	20	G56433	FUS-120020-P	12.0	20
G56429	FUS-095028-P	9.5	28	G56434	FUS-120028-P	12.0	28
G56430	FUS-095035-P	9.5	35	G56435	FUS-120035-P	12.0	35
G56431	FUS-095045-P	9.5	45	G56436	FUS-120045-P	12.0	45
G56432	FUS-095055-P	9.5	55	G56438	FUS-120055-P	12.0	55
G34870	FUS-107013-P	10.7	13	G56439	FUS-140020-P	14.0	20
G34871	FUS-107020-P	10.7	20	G56440	FUS-140028-P	14.0	28
G34872	FUS-107028-P	10.7	28	G56441	FUS-140035-P	14.0	35
G34873	FUS-107035-P	10.7	35	G56442	FUS-140045-P	14.0	45
G34874	FUS-107045-P	10.7	45	G56443	FUS-140055-P	14.0	55
G34875	FUS-107055-P	10.7	55				

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: orders@cookmedical.com

Australia:
 Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
 E-mail: cau.custserv@cookmedical.com

