

Trust in the quality  
of the everyday stent.



Universa<sup>®</sup>  
URETERAL STENT SET




# Universa®

URETERAL STENT SET

## Experience Universa.

Universa stents provide temporary internal drainage from the ureteropelvic junction to the bladder. Universa products have premium features and the advantage of a full product line. Consolidated part numbers enable streamlined ordering and provide cost savings.

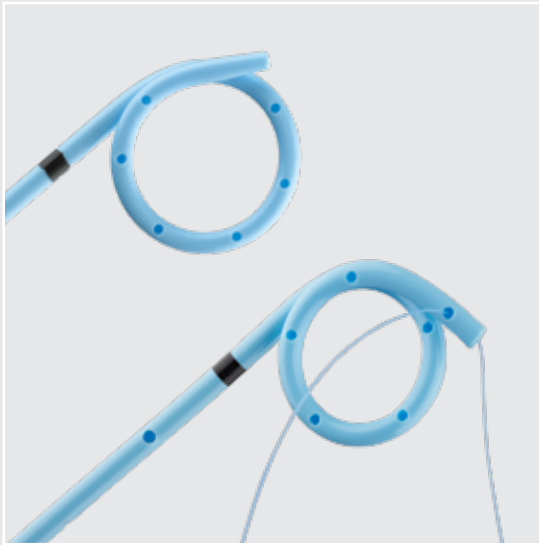

Universa Firm Ureteral Stents



Universa Soft Ureteral Stents

Soft and firm options allow you to choose the appropriate stent for your patient. The Universa firm stent's thermosensitive material is firm at room temperature. When warmed to body temperature, the stiffness of the stent tubing decreases by up to 42%.

## Features and benefits




Graduation marks to visualize stent advancement and placement



Tether (monofilament or braided) to ease repositioning and removal



Pigtail straightener to simplify advancement over a wire guide

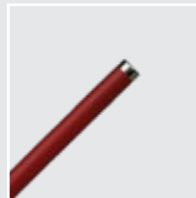

Radiopaque tip on pusher to enable precise positioning under fluoroscopy



French size and length to expedite stent exchanges



Box packaging to provide convenient storage and product identification

## Options

- Stent and positioner - No wire guide
- Set - Stent, positioner, and wire guide
- Hydrophilic - Universa firm hydrophilic, Universa soft hydrophilic
- Nonhydrophilic - Universa soft only

## Universa® Firm Ureteral Stent Set with Hydrophilic Coating and Braided Tether

Order Number	Reference Part Number	Fr	Length cm
<b>Set</b>			
G49862	UFH-518	5.0	18
G49863	UFH-520	5.0	20
G49864	UFH-522	5.0	22
G49865	UFH-524	5.0	24
G49866	UFH-526	5.0	26
G49867	UFH-528	5.0	28
G49868	UFH-530	5.0	30
G49869	UFH-500	5.0	22-32
G49870	UFH-618	6.0	18
G49871	UFH-620	6.0	20
G49872	UFH-622	6.0	22
G49873	UFH-624	6.0	24
G49874	UFH-626	6.0	26
G49875	UFH-628	6.0	28
G49876	UFH-630	6.0	30
G49877	UFH-600	6.0	22-32
G49878	UFH-718	7.0	18
G49879	UFH-720	7.0	20
G49880	UFH-722	7.0	22
G49881	UFH-724	7.0	24
G49882	UFH-726	7.0	26
G49883	UFH-728	7.0	28
G49884	UFH-730	7.0	30
G49885	UFH-700	7.0	22-32
G49886	UFH-818	8.0	18
G49887	UFH-820	8.0	20
G49888	UFH-822	8.0	22
G49889	UFH-824	8.0	24
G49890	UFH-826	8.0	26
G49891	UFH-828	8.0	28
G49892	UFH-830	8.0	30
G49893	UFH-800	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Order Number	Reference Part Number	Fr	Length cm
<b>Universa Firm Ureteral Stent</b>			
G49894	UFH-518-R	5.0	18
G49895	UFH-520-R	5.0	20
G49896	UFH-522-R	5.0	22
G49897	UFH-524-R	5.0	24
G49898	UFH-526-R	5.0	26
G49899	UFH-528-R	5.0	28
G49900	UFH-530-R	5.0	30
G49901	UFH-500-R	5.0	22-32
G49902	UFH-618-R	6.0	18
G49903	UFH-620-R	6.0	20
G49904	UFH-622-R	6.0	22
G49905	UFH-624-R	6.0	24
G49906	UFH-626-R	6.0	26
G49907	UFH-628-R	6.0	28
G49908	UFH-630-R	6.0	30
G49909	UFH-600-R	6.0	22-32
G49910	UFH-718-R	7.0	18
G49911	UFH-720-R	7.0	20
G49912	UFH-722-R	7.0	22
G49913	UFH-724-R	7.0	24
G49914	UFH-726-R	7.0	26
G49915	UFH-728-R	7.0	28
G49916	UFH-730-R	7.0	30
G49917	UFH-700-R	7.0	22-32
G49918	UFH-818-R	8.0	18
G49919	UFH-820-R	8.0	20
G49920	UFH-822-R	8.0	22
G49921	UFH-824-R	8.0	24
G49922	UFH-826-R	8.0	26
G49923	UFH-828-R	8.0	28
G49924	UFH-830-R	8.0	30
G49925	UFH-800-R	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

## Universa® Firm Ureteral Stent Set with Hydrophilic Coating and Monofilament Tether

Order Number	Reference Part Number	Fr	Length cm
<b>Set</b>			
G23428	UFH-518-T1	5.0	18
G23429	UFH-520-T1	5.0	20
G23430	UFH-522-T1	5.0	22
G23431	UFH-524-T1	5.0	24
G23432	UFH-526-T1	5.0	26
G23433	UFH-528-T1	5.0	28
G23434	UFH-530-T1	5.0	30
G23512	UFH-500-T1	5.0	22-32
G23435	UFH-618-T1	6.0	18
G23436	UFH-620-T1	6.0	20
G23437	UFH-622-T1	6.0	22
G23438	UFH-624-T1	6.0	24
G23439	UFH-626-T1	6.0	26
G23440	UFH-628-T1	6.0	28
G23441	UFH-630-T1	6.0	30
G23244	UFH-600-T1	6.0	22-32
G23442	UFH-718-T1	7.0	18
G23443	UFH-720-T1	7.0	20
G23444	UFH-722-T1	7.0	22
G23445	UFH-724-T1	7.0	24
G23446	UFH-726-T1	7.0	26
G23447	UFH-728-T1	7.0	28
G23243	UFH-730-T1	7.0	30
G23319	UFH-700-T1	7.0	22-32
G23448	UFH-818-T1	8.0	18
G23449	UFH-820-T1	8.0	20
G23450	UFH-822-T1	8.0	22
G23451	UFH-824-T1	8.0	24
G23452	UFH-826-T1	8.0	26
G23453	UFH-828-T1	8.0	28
G23454	UFH-830-T1	8.0	30
G23320	UFH-800-T1	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Order Number	Reference Part Number	Fr	Length cm
<b>Stent and Positioner</b>			
G23242	UFH-518-RT1	5.0	18
G23395	UFH-520-RT1	5.0	20
G23396	UFH-522-RT1	5.0	22
G23397	UFH-524-RT1	5.0	24
G23398	UFH-526-RT1	5.0	26
G23399	UFH-528-RT1	5.0	28
G23400	UFH-530-RT1	5.0	30
G23312	UFH-500-RT1	5.0	22-32
G23401	UFH-618-RT1	6.0	18
G23402	UFH-620-RT1	6.0	20
G23404	UFH-622-RT1	6.0	22
G23405	UFH-624-RT1	6.0	24
G23406	UFH-626-RT1	6.0	26
G23407	UFH-628-RT1	6.0	28
G23408	UFH-630-RT1	6.0	30
G23313	UFH-600-RT1	6.0	22-32
G23411	UFH-718-RT1	7.0	18
G23412	UFH-720-RT1	7.0	20
G23413	UFH-722-RT1	7.0	22
G23415	UFH-724-RT1	7.0	24
G23416	UFH-726-RT1	7.0	26
G23417	UFH-728-RT1	7.0	28
G23418	UFH-730-RT1	7.0	30
G23314	UFH-700-RT1	7.0	22-32
G23419	UFH-818-RT1	8.0	18
G23420	UFH-820-RT1	8.0	20
G23421	UFH-822-RT1	8.0	22
G23422	UFH-824-RT1	8.0	24
G23423	UFH-826-RT1	8.0	26
G23424	UFH-828-RT1	8.0	28
G23425	UFH-830-RT1	8.0	30
G23315	UFH-800-RT1	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

## Universa® Soft Ureteral Stent Set with Braided Tether

Order Number	Reference Part Number	Fr	Length cm
<b>Set*</b>			
G53145	US-518	5.0	18
G53672	US-520	5.0	20
G53674	US-522	5.0	22
G53675	US-524	5.0	24
G53676	US-526	5.0	26
G53677	US-528	5.0	28
G53678	US-530	5.0	30
G53687	US-500	5.0	22-32
G53689	US-618	6.0	18
G53690	US-620	6.0	20
G53691	US-622	6.0	22
G53692	US-624	6.0	24
G53693	US-626	6.0	26
G53694	US-628	6.0	28
G53695	US-630	6.0	30
G53703	US-600	6.0	22-32
G53707	US-718	7.0	18
G53708	US-720	7.0	20
G53709	US-722	7.0	22
G53710	US-724	7.0	24
G53711	US-726	7.0	26
G53712	US-728	7.0	28
G53713	US-730	7.0	30
G53706	US-700	7.0	22-32
G53722	US-818	8.0	18
G53723	US-820	8.0	20
G53724	US-822	8.0	22
G53725	US-824	8.0	24
G53726	US-826	8.0	26
G53727	US-828	8.0	28
G53728	US-830	8.0	30
G53721	US-800	8.0	22-32

\*Nonhydrophilic coated

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Order Number	Reference Part Number	Fr	Length cm
<b>Stent and Positioner*</b>			
G53680	US-518-R	5.0	18
G53681	US-520-R	5.0	20
G53682	US-522-R	5.0	22
G53683	US-524-R	5.0	24
G53684	US-526-R	5.0	26
G53685	US-528-R	5.0	28
G53686	US-530-R	5.0	30
G53688	US-500-R	5.0	22-32
G53696	US-618-R	6.0	18
G53697	US-620-R	6.0	20
G53698	US-622-R	6.0	22
G53699	US-624-R	6.0	24
G53700	US-626-R	6.0	26
G53701	US-628-R	6.0	28
G53702	US-630-R	6.0	30
G53704	US-600-R	6.0	22-32
G53714	US-718-R	7.0	18
G53715	US-720-R	7.0	20
G53716	US-722-R	7.0	22
G53717	US-724-R	7.0	24
G53718	US-726-R	7.0	26
G53719	US-728-R	7.0	28
G53720	US-730-R	7.0	30
G53705	US-700-R	7.0	22-32
G53729	US-818-R	8.0	18
G53731	US-820-R	8.0	20
G53732	US-822-R	8.0	22
G53734	US-824-R	8.0	24
G53735	US-826-R	8.0	26
G53736	US-828-R	8.0	28
G53146	US-800-R	8.0	22-32

\*Nonhydrophilic coated

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

## Universa® Soft Ureteral Stent Set with Monofilament Tether

Order Number	Reference Part Number	Fr	Length cm
<b>Set</b>			
G23483	US-518-T1	5.0	18
G23484	US-520-T1	5.0	20
G23485	US-522-T1	5.0	22
G23486	US-524-T1	5.0	24
G23487	US-526-T1	5.0	26
G23488	US-528-T1	5.0	28
G23489	US-530-T1	5.0	30
G23329	US-500-T1	5.0	22-32
G23490	US-618-T1	6.0	18
G23491	US-620-T1	6.0	20
G23492	US-622-T1	6.0	22
G23493	US-624-T1	6.0	24
G23494	US-626-T1	6.0	26
G23495	US-628-T1	6.0	28
G23496	US-630-T1	6.0	30
G23330	US-600-T1	6.0	22-32
G23497	US-718-T1	7.0	18
G23498	US-720-T1	7.0	20
G23499	US-722-T1	7.0	22
G23500	US-724-T1	7.0	24
G23501	US-726-T1	7.0	26
G23502	US-728-T1	7.0	28
G23503	US-730-T1	7.0	30
G23331	US-700-T1	7.0	22-32
G23504	US-818-T1	8.0	18
G23505	US-820-T1	8.0	20
G23506	US-822-T1	8.0	22
G23507	US-824-T1	8.0	24
G23508	US-826-T1	8.0	26
G23509	US-828-T1	8.0	28
G23332	US-800-T1	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Order Number	Reference Part Number	Fr	Length cm
<b>Stent and Positioner</b>			
G23455	US-518-RT1	5.0	18
G23456	US-520-RT1	5.0	20
G23457	US-522-RT1	5.0	22
G23458	US-524-RT1	5.0	24
G23459	US-526-RT1	5.0	26
G23460	US-528-RT1	5.0	28
G23461	US-530-RT1	5.0	30
G23333	US-500-RT1	5.0	22-32
G23462	US-618-RT1	6.0	18
G23463	US-620-RT1	6.0	20
G23464	US-622-RT1	6.0	22
G23465	US-624-RT1	6.0	24
G23466	US-626-RT1	6.0	26
G23467	US-628-RT1	6.0	28
G23468	US-630-RT1	6.0	30
G23334	US-600-RT1	6.0	22-32
G23469	US-718-RT1	7.0	18
G23470	US-720-RT1	7.0	20
G23471	US-722-RT1	7.0	22
G23472	US-724-RT1	7.0	24
G23473	US-726-RT1	7.0	26
G23474	US-728-RT1	7.0	28
G23475	US-730-RT1	7.0	30
G23335	US-700-RT1	7.0	22-32
G23476	US-818-RT1	8.0	18
G23477	US-820-RT1	8.0	20
G23478	US-822-RT1	8.0	22
G23479	US-824-RT1	8.0	24
G23480	US-826-RT1	8.0	26
G23481	US-828-RT1	8.0	28
G23482	US-830-RT1	8.0	30
G23336	US-800-RT1	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

## Universa® Soft Ureteral Stent Set with Hydrophilic Coating and Braided Tether

Order Number	Reference Part Number	Fr	Length cm
<b>Set</b>			
G49762	USH-518	5.0	18
G49931	USH-520	5.0	20
G49932	USH-522	5.0	22
G49933	USH-524	5.0	24
G49934	USH-526	5.0	26
G49935	USH-528	5.0	28
G49936	USH-530	5.0	30
G49937	USH-500	5.0	22-32
G49938	USH-618	6.0	18
G49939	USH-620	6.0	20
G49940	USH-622	6.0	22
G49941	USH-624	6.0	24
G49942	USH-626	6.0	26
G49943	USH-628	6.0	28
G49944	USH-630	6.0	30
G49945	USH-600	6.0	22-32
G49946	USH-718	7.0	18
G49947	USH-720	7.0	20
G49948	USH-722	7.0	22
G49949	USH-724	7.0	24
G49950	USH-726	7.0	26
G49951	USH-728	7.0	28
G49952	USH-730	7.0	30
G49953	USH-700	7.0	22-32
G49954	USH-818	8.0	18
G49955	USH-820	8.0	20
G49956	USH-822	8.0	22
G49957	USH-824	8.0	24
G49958	USH-826	8.0	26
G49959	USH-828	8.0	28
G49960	USH-830	8.0	30
G49961	USH-800	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Order Number	Reference Part Number	Fr	Length cm
<b>Stent and Positioner</b>			
G49962	USH-518-R	5.0	18
G49963	USH-520-R	5.0	20
G49964	USH-522-R	5.0	22
G49965	USH-524-R	5.0	24
G49966	USH-526-R	5.0	26
G49967	USH-528-R	5.0	28
G49968	USH-530-R	5.0	30
G49969	USH-500-R	5.0	22-32
G49970	USH-618-R	6.0	18
G49971	USH-620-R	6.0	20
G49972	USH-622-R	6.0	22
G49973	USH-624-R	6.0	24
G49974	USH-626-R	6.0	26
G49975	USH-628-R	6.0	28
G49976	USH-630-R	6.0	30
G49977	USH-600-R	6.0	22-32
G49978	USH-718-R	7.0	18
G49979	USH-720-R	7.0	20
G49980	USH-722-R	7.0	22
G49981	USH-724-R	7.0	24
G49982	USH-726-R	7.0	26
G49983	USH-728-R	7.0	28
G49984	USH-730-R	7.0	30
G49985	USH-700-R	7.0	22-32
G49986	USH-818-R	8.0	18
G49987	USH-820-R	8.0	20
G49988	USH-822-R	8.0	22
G49989	USH-824-R	8.0	24
G49990	USH-826-R	8.0	26
G49991	USH-828-R	8.0	28
G49992	USH-830-R	8.0	30
G49993	USH-800-R	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



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
## Universa® Soft Ureteral Stent Set with Hydrophilic Coating and Monofilament Tether

Order Number	Reference Part Number	Fr	Length cm
<b>Set</b>			
G23337	USH-518-T1	5.0	18
G23338	USH-520-T1	5.0	20
G23339	USH-522-T1	5.0	22
G23340	USH-524-T1	5.0	24
G23341	USH-526-T1	5.0	26
G23342	USH-528-T1	5.0	28
G23343	USH-530-T1	5.0	30
G23326	USH-500-T1	5.0	22-32
G23344	USH-618-T1	6.0	18
G23345	USH-620-T1	6.0	20
G23346	USH-622-T1	6.0	22
G23347	USH-624-T1	6.0	24
G23348	USH-626-T1	6.0	26
G23349	USH-628-T1	6.0	28
G23350	USH-630-T1	6.0	30
G23241	USH-600-T1	6.0	22-32
G23351	USH-718-T1	7.0	18
G23352	USH-720-T1	7.0	20
G23353	USH-722-T1	7.0	22
G23354	USH-724-T1	7.0	24
G23355	USH-726-T1	7.0	26
G23356	USH-728-T1	7.0	28
G23239	USH-730-T1	7.0	30
G23327	USH-700-T1	7.0	22-32
G23357	USH-818-T1	8.0	18
G23358	USH-820-T1	8.0	20
G23359	USH-822-T1	8.0	22
G23360	USH-824-T1	8.0	24
G23361	USH-826-T1	8.0	26
G23362	USH-828-T1	8.0	28
G23328	USH-800-T1	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Order Number	Reference Part Number	Fr	Length cm
<b>Stent and Positioner</b>			
G23240	USH-518-RT1	5.0	18
G23365	USH-520-RT1	5.0	20
G23366	USH-522-RT1	5.0	22
G23367	USH-524-RT1	5.0	24
G23368	USH-526-RT1	5.0	26
G23369	USH-528-RT1	5.0	28
G23370	USH-530-RT1	5.0	30
G23511	USH-500-RT1	5.0	22-32
G23371	USH-618-RT1	6.0	18
G23372	USH-620-RT1	6.0	20
G23373	USH-622-RT1	6.0	22
G23374	USH-624-RT1	6.0	24
G23376	USH-626-RT1	6.0	26
G23377	USH-628-RT1	6.0	28
G23378	USH-630-RT1	6.0	30
G23316	USH-600-RT1	6.0	22-32
G23379	USH-718-RT1	7.0	18
G23380	USH-720-RT1	7.0	20
G23381	USH-722-RT1	7.0	22
G23382	USH-724-RT1	7.0	24
G23383	USH-726-RT1	7.0	26
G23384	USH-728-RT1	7.0	28
G23385	USH-730-RT1	7.0	30
G23317	USH-700-RT1	7.0	22-32
G23386	USH-818-RT1	8.0	18
G23387	USH-820-RT1	8.0	20
G23388	USH-822-RT1	8.0	22
G23389	USH-824-RT1	8.0	24
G23390	USH-826-RT1	8.0	26
G23391	USH-828-RT1	8.0	28
G23392	USH-830-RT1	8.0	30
G23318	USH-800-RT1	8.0	22-32

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



#### Universa® Stent Soft or Firm

**CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.**

**INDICATIONS FOR USE:** Used for temporary internal drainage from the ureteropelvic junction to the bladder. Ureteral stents have been employed to relieve obstruction in a variety of benign, malignant, and post-traumatic conditions. These stents may be placed using endoscopic, percutaneous, or open surgical techniques. The multi-length stents will accommodate ureters from approximately 22-32cm in length.

**CONTRAINDICATIONS:** There are no known contraindications.

**WARNINGS:** Formation of knots in multi-length stents may occur. This may result in injury to the ureter during removal and/or the need for additional surgical intervention. The presence of a knot should be considered if significant resistance is encountered during attempts at removal.

**CAUTIONS: Intended for one-time use only.** • The Universa® Firm stents must not remain indwelling more than twelve (12) months. The Universa® Soft stents must not remain indwelling more than six (6) months. If the patient's status permits, the stent may be replaced with a new stent. • These stents are not intended as permanent indwelling devices. • Tether should be removed if stent is to remain indwelling longer than 14 days. • Do not force components during removal or replacement. Carefully remove the components if any resistance is encountered. • A pregnant patient must be more closely monitored for possible stent encrustation due to calcium supplements. • Improper handling can seriously weaken the stent. Acute bending or overstressing during placement may result in subsequent separation of the stent at the point of stress after a prolonged indwelling period. Angulation of the wire guide or stent should be avoided. Use of a 0-degree scope lens is recommended. Scopes larger than 21.0 French are suggested. • Individual variations of interaction between stents and the urinary system are unpredictable. • Periodic evaluation via cystoscopic, radiographic, or ultrasonic means is suggested. The stent must be replaced if encrustation hampers drainage. • If problems occur using this device, please call your Cook Urological sales representative or contact our Customer Quality Assurance department at the address/phone listed in the complete **INSTRUCTIONS FOR USE**.

See instructions for use for full product information.

AB\_T\_USS\_REVO

#### Universa® Soft Ureteral Stents and Stent Sets

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

**INTENDED USE:** The Universa Soft Ureteral Stents and Stent Sets are used for temporary internal drainage from the ureteropelvic junction to the bladder. Ureteral stents have been used to relieve obstruction in a variety of benign, malignant and post-traumatic conditions. The stents may be placed using endoscopic, percutaneous or open surgical techniques.

**CONTRAINDICATIONS:** None known

**WARNINGS:** Formation of knots in multi-length stents may occur. This may result in injury to the ureter during removal and/or the need for additional surgical intervention. The presence of a knot should be considered if significant resistance is encountered during attempts at removal. • After use this device may be a potential biohazard. Handle and dispose of in accordance with acceptable medical practices and with applicable local, state and federal laws and regulations.

**PRECAUTIONS:** The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for the reproductive and developmental effects. • The Universa Soft Stents must not remain indwelling more than six months. If the patient's status permits, the stent may be replaced with a new stent. • These stents are not intended as permanent indwelling devices. • **The tether should be removed if the stent is to remain indwelling longer than 14 days.** • Do not force components during removal or replacement. Carefully remove the components if any resistance is encountered. • A pregnant patient must be more closely monitored for possible stent encrustation due to calcium supplements. • Improper handling can seriously weaken the stent. Acute bending or overstressing during placement may result in subsequent separation of the stent at the point of stress after a prolonged indwelling period. • Angulation of the wire guide or stent should be avoided. Use of a 0-degree scope lens is recommended. Scopes larger than 21.0 French are suggested. • Individual variations of interaction between stents and the urinary system are unpredictable. • Ureteral stents should be checked periodically for signs of encrustation and proper function. Periodic checks of the stent by cystoscopic and /or radiographic procedures are recommended at intervals deemed to be appropriate by the physician in consideration of the individual patient's condition and other patient specific factors. The stent is not intended as a permanent indwelling device which should not exceed 180 days. • Use of the device should be based upon consideration of risk-benefit factors as they apply to your patient.

**POTENTIAL ADVERSE EVENTS:** Complications of ureteral stent placement are documented. These complications include, but are not limited to: • Extravasations • Occlusion • Migration • Hemorrhage • Sepsis • Perforation of the urinary tract • Peritonitis • Encrustation • Urinary tract infection • Loss of renal function.

See instructions for use for full product information.

T\_USUS\_REVO

## Customer Service

EU Website: [cookmedical.eu](http://cookmedical.eu)  
EDI: [cookmedical.eu/edi](http://cookmedical.eu/edi)  
Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
Belgium: +32 27001702, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
Iceland: +354 8007615, [IS.orders@cookmedical.com](mailto:IS.orders@cookmedical.com)  
Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
Poland: +48 223060159, [pl.orders@cookmedical.com](mailto:pl.orders@cookmedical.com)  
Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

USA Website: [cookmedical.com](http://cookmedical.com)  
EDI: [cookmedical.com/edi.do](http://cookmedical.com/edi.do)  
**Americas:**  
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
E-mail: [orders@cookmedical.com](mailto:orders@cookmedical.com)  
**Australia:**  
Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283  
E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



CC, URO-8.5X11