


# BIGopsy<sup>®</sup> setup: 3 simple steps


After you backload the BIGopsy through the working channel of the ureteroscope, secure the handle to the forceps by following the instructions below. To properly grip the handle, place the thumb of your dominant hand in the thumb ring and control the spool with the index finger and middle finger of the same hand.

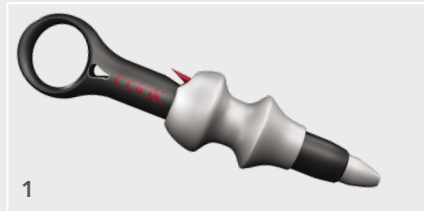

Use a finger from your free hand to depress the button. With your dominant hand, retract the spool over the button. Release the button to secure the handle in the retracted position.



With your free hand, feed the forceps shaft through the handle until it passes completely through the viewing window of the handle.

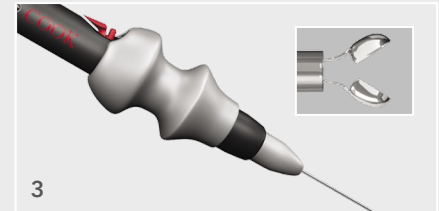

Hold the spool firmly in place while you press the button to release the spool. Ease the spool into its original position to secure the forceps shaft in the handle.

**BIGopsy<sup>®</sup>**  
BACKLOADING BIOPSY FORCEPS



Please see product risk information in the IFU at [cookmedical.eu](http://cookmedical.eu)