


Overcome the challenges of male urethral dilation.




S~Curve®
URETHRAL DILATOR SET

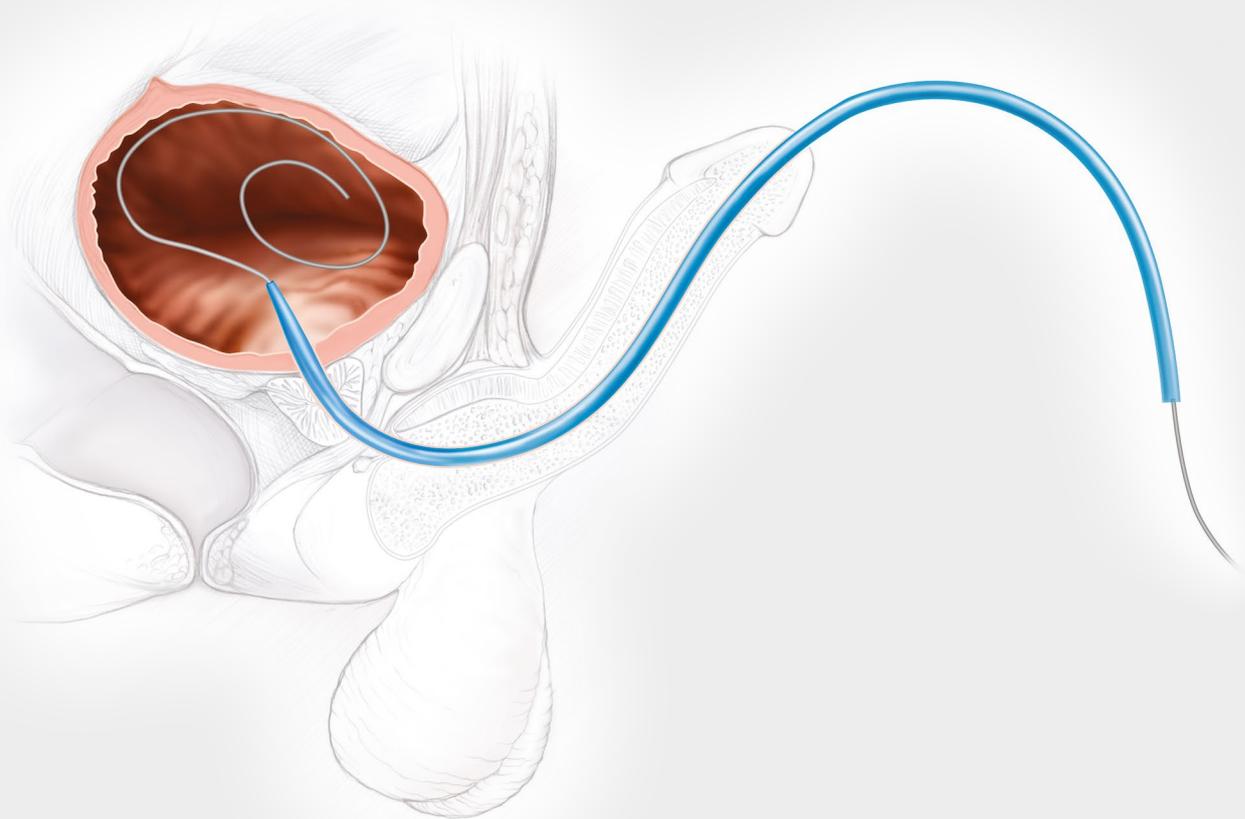

S~Curve[®]


URETHRAL DILATOR SET

S~Curve shape helps traverse and support the natural curvature of the male urethra. The dilators are shaped in an S~Curve formation and have a tapered tip.

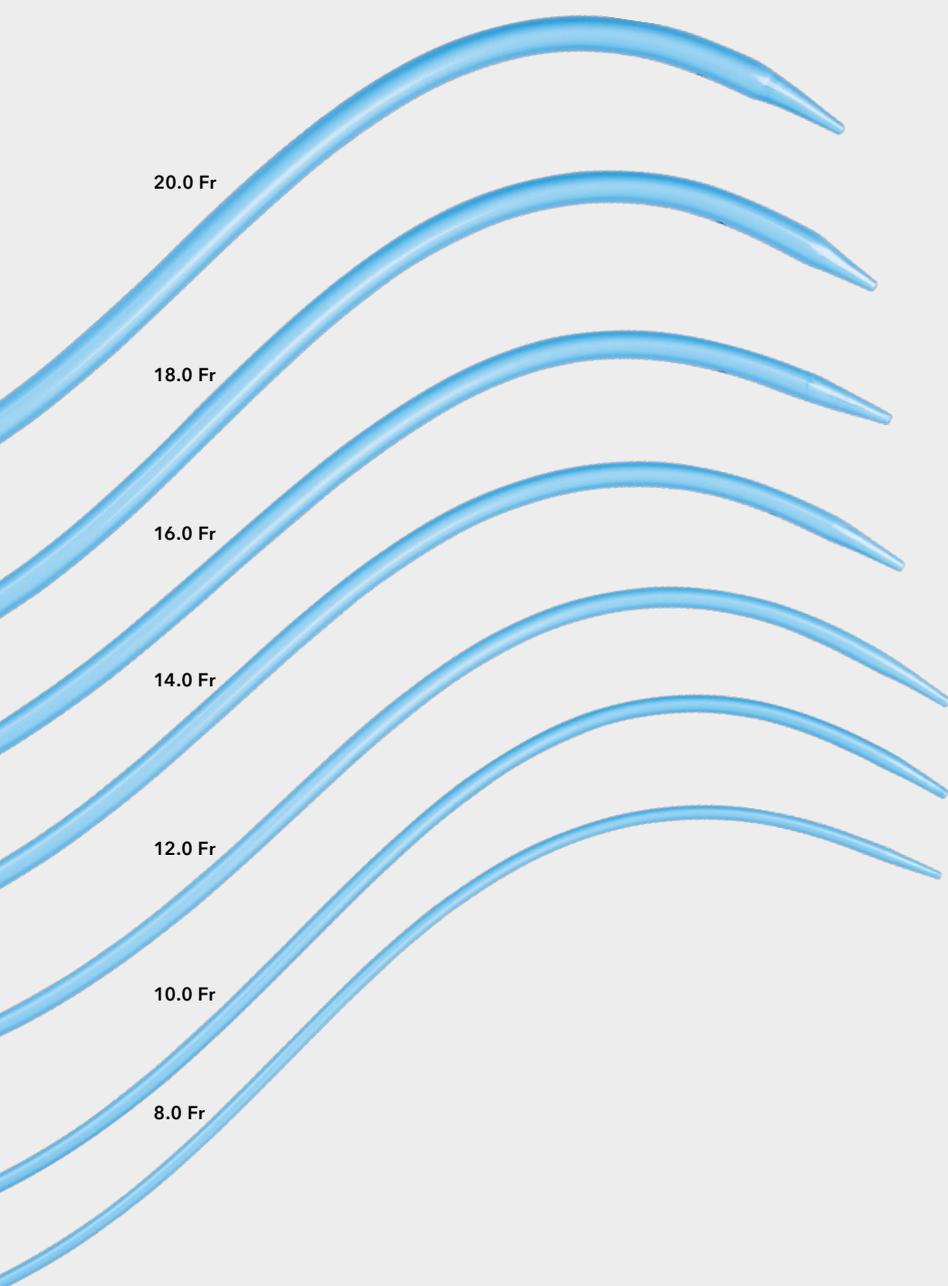

Features and benefits

- AQ[®] hydrophilic coating is applied to the surface of the dilator to enhance lubricity of device for ease of entry to the access site.
- The lumen allows you to advance the dilator over a previously positioned wire guide.
- The ported tip allows urine to drain in order to confirm proper placement.
- The tray is designed for convenience.
- The reservoir allows the AQ hydrophilic coating to hydrate and activate while the dilator is still on the tray.
- The embossed size markings allow you to identify and select sizes.

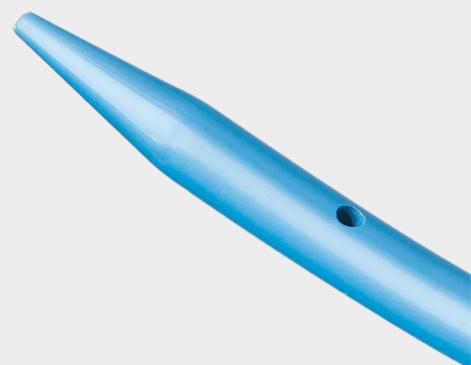

Note: Before use, immerse the AQ dilator in sterile water or isotonic saline to allow the hydrophilic surface to absorb liquid and become lubricious. This will facilitate placement under standard conditions.

Step 1



Position the wire guide

Under direct vision or fluoroscopic control, position a **wire guide** through the urethra and into the bladder.


Step 2 – Traditional method or alternative method



Introduce the smallest appropriately sized dilator (use one of the two methods that follow)

Traditional method:


- Hold the penis in an **upward position**.
- Orient the dilator so the **French size on the end without a tapered tip** faces **upward**, and introduce the dilator over the previously placed wire guide.

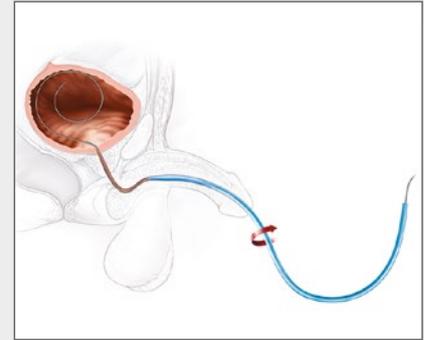

Alternative method:

- Hold the penis in a **downward position**.
- Orient the dilator so the **French size on the end without a tapered tip** faces **downward**, and introduce the dilator over the previously placed wire guide.
- Advance the dilator to the level of the **bulbar urethra**.
- With the dilator positioned at the level of the bulbar urethra, rotate the dilator **180 degrees**, so the dilator is now oriented with the French size stamp facing **upward**.

Step 3

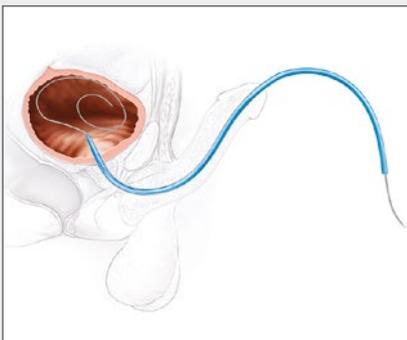

Advance the dilator

Continue to advance the dilator into the bladder. **Urine drainage will confirm proper placement.**

Step 4

Progress to the desired size

While maintaining the wire guide position, remove the dilator currently in place and replace it with the next appropriately sized dilator.

Repeat steps 2-3 as needed, progressing from the smallest to the largest appropriately sized dilator until the desired urethral dilation is achieved.

Please refer to the product's Instructions for Use (IFU) for full prescribing information, warnings, precautions, contraindications, and potential adverse events.



S~Curve® URETHRAL DILATOR SET

The S~Curve Urethral Dilator Set is used for dilation of the male urethra in adult patients by retrograde insertion of the devices.



Order Number	Reference Part Number	Fr	Length cm
S~Curve® Urethral Dilator Set			
G32789	073701-CD	8.0-20.0	37
S~Curve® Urethral Dilators			
G46249	073722-CD	22.0	37
G46250	073724-CD	24.0	37
G46251	073726-CD	26.0	37

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support & Distribution for details. Please see product risk information in the IFU at cookmedical.eu.

Customer Service

EU website: cookmedical.eu
 EDI: cookmedical.eu/edi
 Distributors: +353 61239240, emea.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001702, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 21612723770, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Iceland: +354 8007615, is.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Poland: +48 223060159, pl.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

USA website: cookmedical.com
 EDI: cookmedical.com/edi.do

Americas:
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 Email: customersupport@cookmedical.com

Australia:
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
 Email: cau.custserv@cookmedical.com



CC, URO-A4