


**Secure and extract
small stone fragments.**

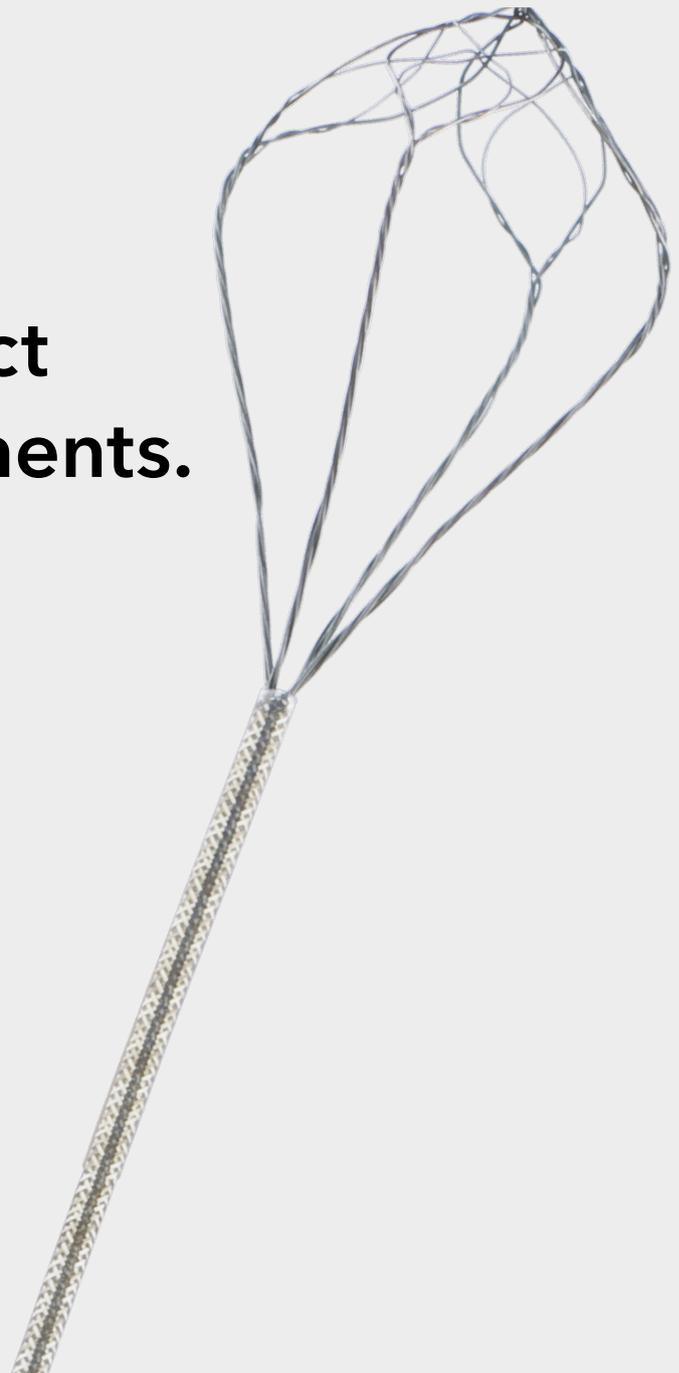

NCompass[®]
NITINOL STONE EXTRACTOR



NCompass®


NITINOL STONE EXTRACTOR

The NCompass® Nitinol Stone Extractor, featuring multiwire geometry, is designed to capture a wide range of stone sizes.


The unique multiwire geometry provides a traditional 4-wire basket design at full deployment. Partial closure redesigns the basket to provide the weave of a 16-wire basket. The duality of the NCompass extractor allows both routine capture and extraction of intact stones. NCompass features a design to capture multiple small fragments after intracorporeal lithotripsy.

Features

Intricate multiwire geometry	Dual-basket design: a standard 4-wire basket at full deployment; a 16-wire basket during partial closure. The unique multiwire geometry allows the basket to completely encompass stones for secure capture and extraction, and it may help prevent stone migration.
Tipless design	The tipless basket design follows in the tradition of the NCircle® basket.
Nitinol alloy	Resilient memory characteristics retain their preformed shape when the basket is deployed. Kink resistance creates a durable basket. Basket flexibility allows capture and release of stones.
Flexible sheath	The flexible sheath permits continuous deflection of the ureteroscope during stone manipulation.
1.7 Fr sheath option	Available in 2.2 Fr and smaller 1.7 Fr sheath options. The sheath size and flexibility allow improved irrigation flow through the working channel of an endoscope.

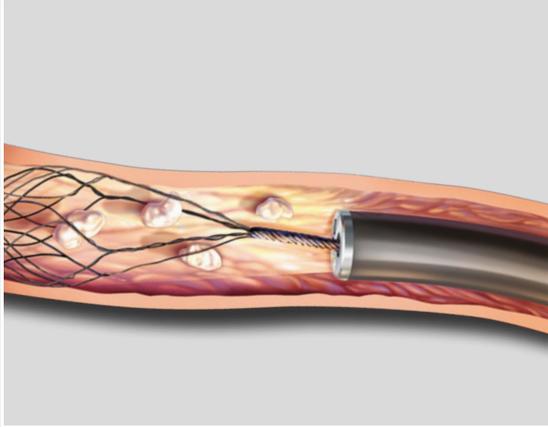


1. At complete deployment the NCompass stone extractor allows for stone entry.

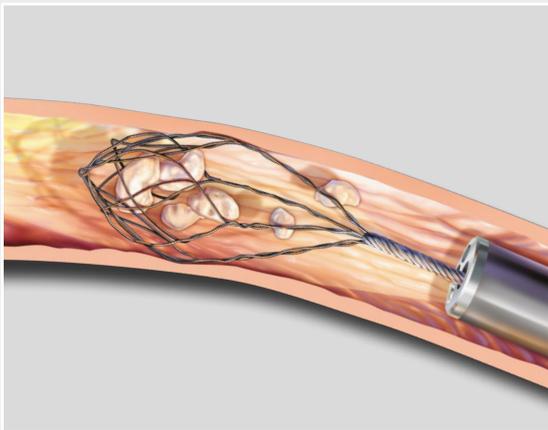

2. The multiwire area at the distal portion of the basket captures the stones as the basket is withdrawn through the ureter. This capture may prevent stone migration.



3. As the NCompass stone extractor is gradually closed, the multiwire geometry allows the basket to completely encompass the stones for secure capture and extraction.

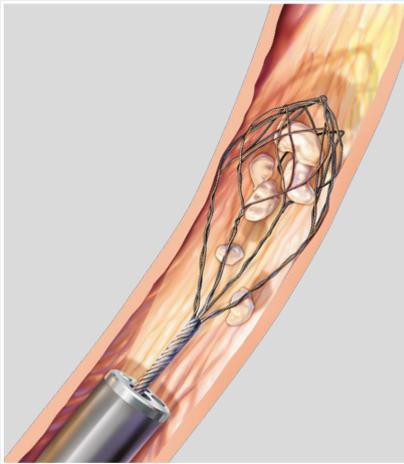

NCompass® NITINOL STONE EXTRACTOR

Used for stone manipulation and removal in the urinary tract through the working channel of an endoscope during cystoscopic or ureteroscopic procedures by actuating a nitinol wire basket.

Order Number	Reference Part Number	Fr	Length cm	Basket Diameter cm	Basket Configuration	Tip Configuration
G46027	NCT4-017115	1.7	115	0.9	4-wire, 16-wire	tipless
G32701	NCT4-024115	2.4	115	0.9	4-wire, 16-wire	tipless
G32863	NCT4-024115-MB	2.4	115	1.5	4-wire, 16-wire	tipless

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Support & Distribution for details. Please see product risk information in the IFU at cookmedical.eu.

Customer Service

EU website: cookmedical.eu
 EDI: cookmedical.eu/edi
 Distributors: +353 61239240, emea.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001702, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 21612723770, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Iceland: +354 8007615, is.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Poland: +48 223060159, pl.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

USA website: cookmedical.com
 EDI: cookmedical.com/edi.do

Americas:
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 Email: customersupport@cookmedical.com

Australia:
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
 Email: cau.custserv@cookmedical.com

